

# Cleaning and Disinfecting the ComfortLite™ 2 Mask



## CLEANING

Clean the mask prior to using any of the disinfection or sterilization methods listed below.

- Remove the headgear and spacer, if appropriate.
- Soak the parts in a commercially available enzymatic cleaner (e.g., Klenzyme™).
- Clean the mask with a soft bristle brush in a solution of cool tap water and a commercially available anionic detergent (e.g., Manu Klenz™). Do not use cleaning products that contain conditioners or moisturizers because they will leave a residue.
- Rinse thoroughly under cool running tap water. Air dry.
- If adhering substances have not been adequately removed, replace the mask.



## DISINFECTION/STERILIZATION PROCESS

### PASTEURIZATION

TEMPERATURE	EXPOSURE TIME	RESULT	NO. CYCLES TESTED	NOTES
70° C (performance tested at 75° C)	30 minutes	High level disinfection	30	<ul style="list-style-type: none"> <li>• Inspect the mask after processing. If any components are damaged, replace the mask.</li> <li>• Slight discoloration of the mask cushion after processing is normal.</li> </ul>
90° C	1 minute			

### CIDEX® 2.4%

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
45 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> <li>• Inspect the mask after processing. If any components are damaged, replace the mask.</li> <li>• Slight discoloration of the mask cushion after processing is normal.</li> </ul>

### CIDEX® O.P.A.

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
12 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> <li>• Inspect the mask after processing. If any components are damaged, replace the mask.</li> <li>• Slight discoloration of the mask cushion after processing is normal.</li> </ul>

**WAVICIDE®**

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
45 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> <li>• Inspect the mask after processing. If any components are damaged, replace the mask.</li> <li>• Slight discoloration of the mask cushion after processing is normal.</li> </ul>

**CONTROL 3®**

EXPOSURE TIME	RINSE	RESULT	NO. CYCLES TESTED	NOTES
10 minutes	Per manufacturer's instruction	High level disinfection	30	<ul style="list-style-type: none"> <li>• Inspect the mask after processing. If any components are damaged, replace the mask.</li> <li>• Slight discoloration of the mask cushion after processing is normal.</li> </ul>

Contact Customer Service to obtain mask disinfecting instructions for the ComfortLite 2 (#1032071).

Contact Customer Service to obtain mask disinfecting instructions for other Respiroics masks (#1011138).

Customer Service: 1-800-345-6443 • 724-387-4000

Respiroics Europe: +33 1 47 52 30 00

Respiroics Asia Pacific: +81-3-5280-9611



**RESPIRONICS®**  
Deutschland  
Gewerbestrasse 17  
82211 Herrsching Germany



©2006 Respiroics, Inc., and its affiliates. All rights reserved.

1028235  
JW 01/11/06