



RESPIRONICS

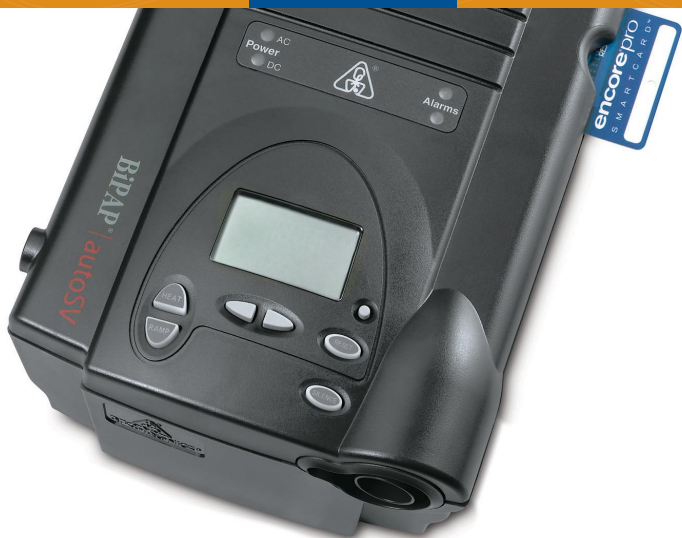

BiPAP[®] autoSV[™] Sleep Therapy System SET UP GUIDE

Patient Room

Control Room

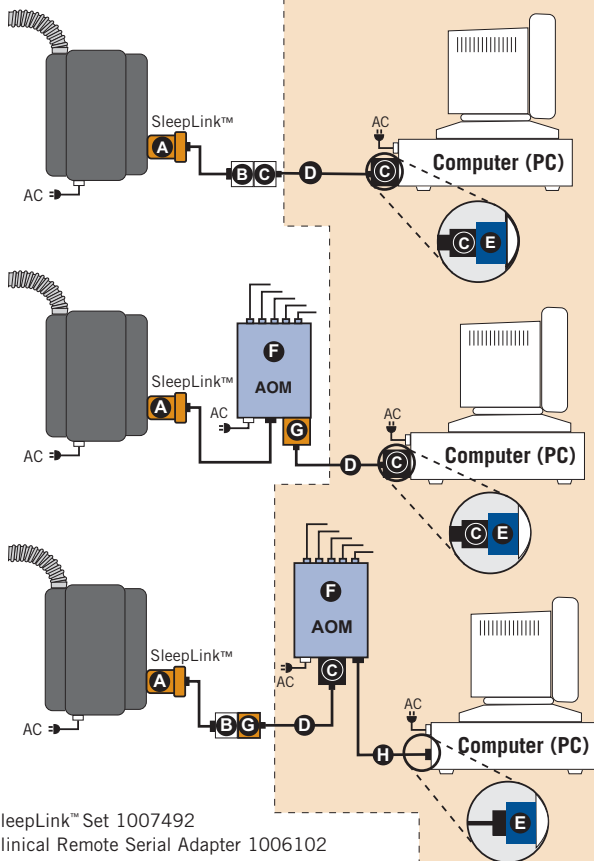

A - SleepLink™ Set 1007492

B - Clinical Remote Serial Adapter 1006102

C - DB9 to RJ45 Crossover Adapter female 1023456*

D - CAT5 Cables (10, 50, 100 feet) H3150-XX*

E - USB to Serial Adapter 1022895

F - Analog Output Module (AOM) System 1006966

G - RJ45 to DB9 Comm Adapter female 1020656*

H - DB9 to DB9 Cable 72" 1000814

*These components can be ordered as a set.

SETTINGS

Some definitions

EPAP	End Expiratory Positive Airway Pressure
IPAP _{min}	Minimum Inspiratory Pressure
IPAP _{max}	Maximum Inspiratory Pressure
BPM	Back-Up Rate
Ti	Inspiratory Time

Settings

EPAP: 4 to 25 cm H₂O

- Should be adjusted to treat Obstructive Component

IPAP_{min}: EPAP to 30 cm H₂O

- Equal to EPAP for CPAP to treat OSA
- Add 2–3 cm above EPAP for comfort

IPAP_{max}: IPAP_{min} to 30 cm H₂O

Set to treat Obstructive Component

- EPAP = IPAP_{min}: CPAP
- EPAP < IPAP_{min}: BiPAP®

Automatic Pressure Support Maximum


- RATE**
- OFF
 - Auto
 - 4-30 bpm

Allows for clinician adjustments to back-up rate

DEFAULT SETTINGS

- EPAP: 5 cm H₂O
- IPAP_{min}: 5 cm H₂O
- IPAP_{max}: 15 cm H₂O
- RATE: Auto

HOW IT WORKS

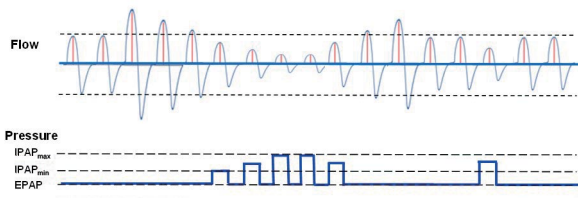


BiPAP® autoSV™ SET UP GUIDE

> If setup appears on the screen, the machine is unlocked. Go directly to frames 1 and 2



> To unlock the machine:





1) Hold  &  together for more than 2 seconds

2) To keep the machine unlocked, press  several times to reach the unlock screen. "SETUP" & "0" flash on the screen:



Set to 1 by pressing 

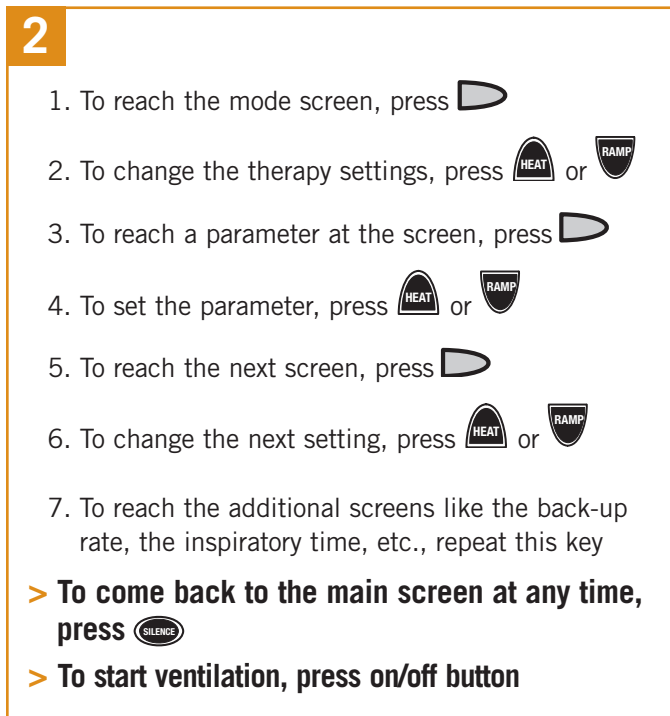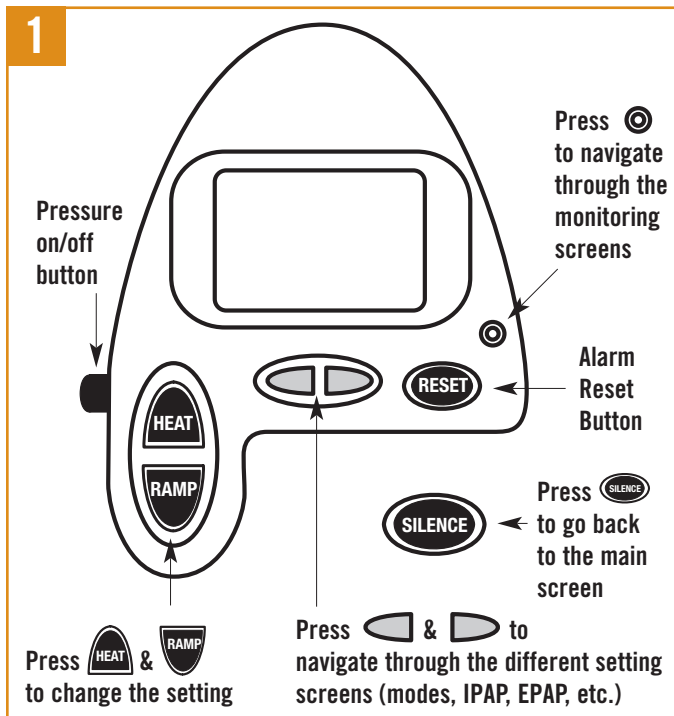

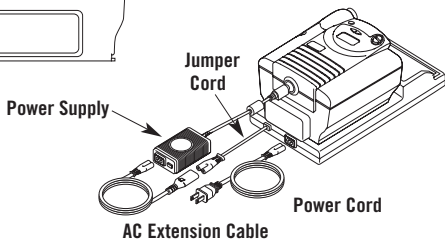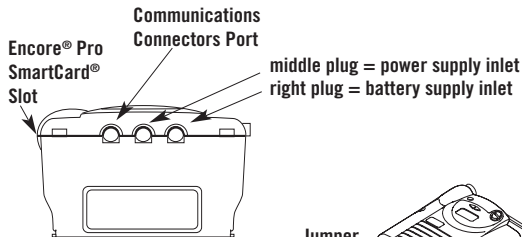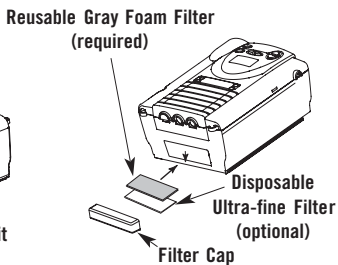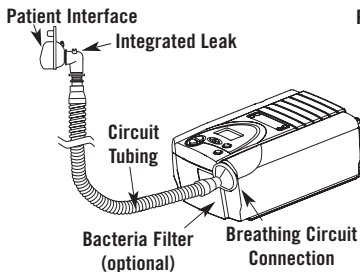

RESPIRONICS®

Customer Service: 800-345-6443 or 724-387-4000
 Respironics Europe, Africa, Middle East: +33-1-47-52-30-00
 Respironics Asia Pacific: +852-3194-2280
www.respironics.com

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician.

Covered by one or more patents. Other patents pending.

©2007 Respironics, Inc. and its affiliates. All rights reserved.

Hoech KW 5/7/07 MCI 4101056 PN 1043629